
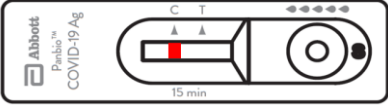


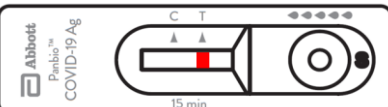

Employee Operator ID (employee ID#):	Name Last/First:
Location:	Educational Designate:

Complete the True/False quiz, a minimum of 80% is required for successful pass

	Question	True/ False
1	Panbio COVID-19 Ag Rapid Test kits must be brought to room temperature (15-30°C) for 30 minutes prior to performing a test.	T
2	Samples must be tested immediately after collection or stored in an extraction tube with buffer at room temperature for up to 2 hours.	T
3	Sample extraction is performed by swirling the swab tip in the extraction tube buffer and pushing into the wall at least 3 times.	F
4	2 drops of extracted specimen will be dispensed onto the test device.	F
5	Any specimen type (oral swab, throat swab, or nasopharyngeal swab) is acceptable to use with the Panbio COVID-19 Ag Rapid Test device.	F
6	Reading test results earlier than 15 minutes or later than 20 minutes may give incorrect results.	T
7	This image indicates a negative test result. 	T
8	This image indicates a positive test result. 	F
9	A positive test result will be reported as positive, the health care provider must follow AHS and Alberta Health guidelines for isolation, and book confirmatory testing through the AHS portal.	T
10	A negative test result will be reported as negative and no further action/follow-up is required.	T

Score: ____/10 ____%	Educator/Designate Signature:	Date Completed:
---------------------------	-------------------------------	-----------------

The Education Quiz is to be retained by the healthcare professional/provider (trainee) for proof of completion and competency. Mark as completed on the Education Checklist.